

SYNCHRONIZED RF & HIFEM: HUMAN ADIPOCYTE APOPTOSIS HISTOLOGY

INDUCTION OF FAT APOPTOSIS BY A COMBINATION OF SYNCHRONIZED RADIOFREQUENCY AND HIFEM TECHNOLOGY: HUMAN HISTOLOGY STUDY

David J. Goldberg M.D.^{1,2}


1. Skin Laser & Surgery Specialists, Division of Schwieger Dermatology Group, New York, New York, USA

2. Department of Dermatology, Icahn School of Medicine at Mt. Sinai, New York, New York, USA

Published in the Journal of Cosmetic Dermatology, January 2024, DOI: 10.1111/jocd.16197

HIGHLIGHTS

- 9 subjects (23-61 years, BMI = 24.6-34.7 kg/m², skin type I-V) were divided into the **active group (n=6, receiving one HIFEM+RF treatment)** and control group (n=3, sham treatment)
- **Punch biopsies** were obtained at **baseline, 8 hours, and 24 hours** after the treatment to measure the **levels of caspase-3**
- **Caspase-3** is an enzyme initiating the execution phase of apoptosis, as it is responsible for nucleus destruction and DNA fragmentation. **After caspase-3 activation, apoptosis is considered to be inevitable.**
- Immunohistochemical evaluation showed **significant activation of caspase-3** in the active group (p<0.001), indicating an elevated level of apoptosis.
- The apoptotic index increased on average from **7.34 ± 2.34% to 47.01 ± 10.53% on average** 8 hours after treatment (+540%)
- No significant change in the levels of apoptotic cells was observed in the control group

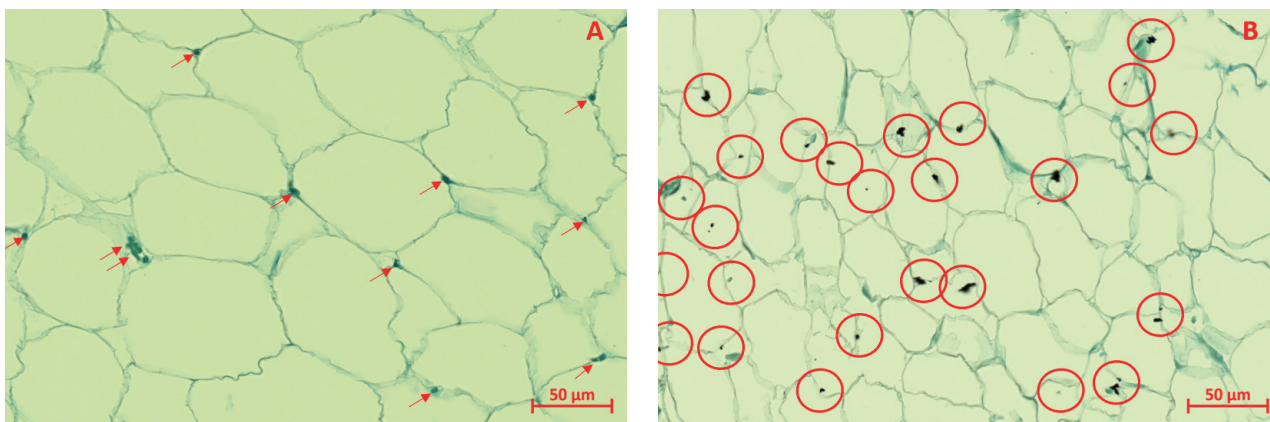

The left side (A) represents untreated adipose tissue with a healthy nucleus (red arrows), while the right side (B) represents adipose tissue 8 hours after the HIFEM+RF treatment with numerous apoptotic nuclei (red circles).